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GBR INSTRUCTIONS FOR USE IPPB System ML REF 002.010

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Authorised use

The IPPB System ML is used for IPPB therapy in conjunction with the Aerolife 2 / Aerolife 3. The IPPB System ML is intended for single use only. When reusing both the patient and the operating personnel could be endangered. The cleaning agent or disinfectant used in the workup may lead to changes in the material properties of the tube system components. The operational safety is no longer given and the manufacturer's liability excluded.

Instructions for use

- Also observe the instructions from Aerolife 2 / Aerolife 3
- Take the hose system from the packaging
- The mouthpiece REF 002 010 006 is to be stuck on the adapter (see assembly instruction)
- The IPPB System ML must be connected to the Aerolife 2 / Aerolife 3. The hoses have three different Diameter and fit only to the appropriate connections of the same size on the device (see assembly instruction)
- Use nebuliser horizontally
- Disposal in accordance with current legal regulations

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Montageanleitung IPPB System ML REF 002.010

Assembly instruction IPPB System ML REF 002.010